LuxmedIQ

VAGINALER PH TEST APPLIKATOR
PH-S23H

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LuxmedIQ

VAGINAL PH TEST DEVICE
PH-S23H

For self-testing in vitro diagnostic use.

INTENDED USE

The Vaginal pH Test Device is a rapid test for measuring the pH value of vaginal fluid samples. It is a semi-quantitative, non-automated test designed for self-testing and in-vitro diagnostic use. The test helps determine whether vaginal pH levels are within the normal range and serves as an aid in diagnosing bacterial vaginosis.

This test is intended for women of menstruating age who experience symptoms such as vaginal itching, burning, or an abnormal color, odor, or excessive vaginal discharge.

INTRODUCTION

Each package contains hygienically packaged pH test applicators for measuring the vaginal pH (refer to the box for details on package size), along with a set of usage instructions and a pH color scale.

A vaginal pH level between 3.8 and 4.5 is essential for maintaining the body's natural defense system that protects the vagina*. This protective mechanism helps prevent the colonization of harmful pathogens and reduces the risk of vaginal infections. A healthy vaginal flora is the most important and natural safeguard against vaginal health issues.

Various factors regulate the composition and balance of the vaginal microbiome, ensuring its protective functions. One key factor is lactobacilli. These beneficial bacteria help break down glycogen released from the upper vaginal wall into glucose, which is then converted into lactic acid. As a result, an acidic vaginal environment develops, maintaining a pH level between 3.8 and 4.5. This acidic condition provides effective protection against infections, as most pathogenic bacteria cannot thrive in such an environment.

The physiological vaginal pH level depends on the quantitative presence of lactic acid[†]. However, vaginal pH levels can fluctuate due to various factors. Possible causes of alterations in vaginal pH levels include:

- Bacterial vaginosis (an imbalance in the vaginal microbiome)
- Mixed bacterial infections
- Sexually transmitted infections (STIs)
- Premature rupture of fetal membranes
- Estrogen deficiency
- Postoperative wound infections
- Overuse of intimate hygiene products
- Antibiotic treatment

Everyone knows that vinegar and lemon juice are acidic, while soap suds are alkaline. The degree of acidity or alkalinity is measured using the pH scale, which ranges from 1 to 14. Pure water has a neutral pH of 7.0, meaning it is neither acidic nor alkaline.

A physiologically normal vaginal pH (ranging from 3.8 to 4.5) plays a crucial role in the body's natural defense system, helping to protect the vagina and maintain a healthy vaginal flora. However, when the vaginal pH rises above 4.5, it can disrupt this balance, leading to discomfort and a reduced quality of life for affected women:

- Poses a significant health risk
- Increases the likelihood of premature delivery

A disruption in the vaginal microecological balance (e.g., dysbiosis) is often associated with an increase in pH levels. Vaginal pH levels higher than 4.5 can lead to various gynecological issues and complications.

Bacterial vaginosis (BV) is diagnosed in approximately 15–20% of all pregnant women, increasing the risk of:

- Premature birth
- Chorioamnionitis (amnionitis) – Infection of the amniotic sac and surrounding membranes
- Premature rupture of the fetal membranes
- Fever during labor
- Postpartum endometritis – Infection of the uterine lining after childbirth
- Bacterial infection in the fetus

By regularly self-testing their vaginal pH levels, pregnant women can reduce the risk of infections. According to Prof. E. Saling, measuring vaginal pH is an essential part of the program for preventing premature births.

WHEN IS THE BEST TIME TO USE A PH TESTING DEVICE?

- In cases of bacterial vaginosis (an imbalance in the vaginal microbiota)
- Suspected dysbiosis
- After treatment for an infection (e.g., antibiotic therapy)
- As part of routine preventive medical check-ups
- During pregnancy
- In cases of a known risk of preterm birth

- After sexual contact with a new partner
- When experiencing vaginal symptoms such as burning, itching, an unusual discharge, or a fishy odor
- Before and after surgical procedures involving the vagina

LESS EFFORT – GREATER EFFICIENCY

Regular use of a pH test helps monitor the vaginal microecological balance and helps detect any disturbances, such as dysbiosis.

This test is designed for home use. We recommend performing the test twice a week to check whether your vaginal pH remains within a healthy range between visits to your gynecologist. However, you can test more frequently if needed. This is especially advisable if you have previously experienced vaginal infections during pregnancy or have had a premature birth. Frequent testing is also recommended if you often experience intimate discomfort, such as itching, burning, a fishy odor, or increased discharge.


PRINCIPLE

The Vaginal pH Test Device is a reliable, hygienic, and pain-free method for determining vaginal pH levels. As soon as the pH measurement zone on the applicator comes into contact with vaginal secretions, it changes color. This color change corresponds to a value on the provided pH color scale, which indicates the test result.

The Vaginal pH Test Device is a patent-pending medical product designed for easy and precise self-testing.

The device consists of a vaginal applicator approximately 7.5 cm in length, with an integrated pH measurement strip (MERCK pH indicator rods, pH 4.0 - 7.0, non-blood, 1.09542.0001). It features a round handle area for safe handling and an insertion tube approximately 5 cm long.

At the tip of the insertion tube, there is a small window that houses the pH measurement area. To perform the test, the applicator is inserted about three centimeters into the vagina, and the pH measurement area is gently pressed against the back wall to collect vaginal secretions. Once moistened, the applicator is removed, and the pH level can be read from the color scale.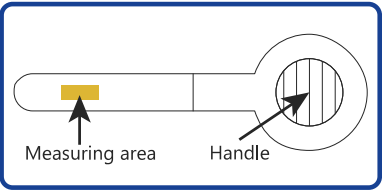

MATERIALS

PROVIDED MATERIALS

- Vaginal pH Test Device
- 1 Desiccant
- 1 pH color scale
- Instruction for Use

REQUIRED MATERIALS (NOT INCLUDED)

- Timer

PRECAUTIONS

- Each pH test applicator is for single use only.
- Use only as intended; not for consumption.
- The applicator measures pH value only and does not detect infections.
- An acidic pH value does not guarantee complete protection against infections. If you experience symptoms despite a normal pH value, consult your doctor.
- Do not use the test after the expiration date (see packaging for details).
- Do not use the test if the foil packaging is damaged.
- Read the instructions carefully before using the test.
- Certain factors may temporarily alter vaginal pH levels and lead to inaccurate results. To ensure reliable measurements, please observe the following time intervals before performing the test:
 - Wait at least 12 hours after sexual activity.
 - Wait at least 12 hours after using vaginal medical products (such as suppositories, creams, or gels).
 - If you are not pregnant, wait 3 to 4 days after the end of your period before testing.
 - Wait at least 15 minutes after urination, as residual urine may affect the test results.
- Do not wash or shower the area immediately before taking the measurement.
- Be aware that urine residue may lead to inaccurate test results.
- Never start any treatment without first discussing the test result with a doctor.
- If the test applicator is not used correctly, it may cause the hymen to tear in women who are not yet sexually active. This is comparable to using a tampon.
- After completing the test, dispose of the applicator according to local regulations.


STORAGE AND STABILITY

- Store the kit at a temperature between 2°C and 30°C until the expiry date printed on the sealed pouch.
- Keep the test in its sealed pouch until use and, ideally, use it within 2 hours after first opening the pouch.
- Protect the kit from humidity.
- Take precautions to prevent contamination of the kit components. Do not use the test if there are signs of microbial contamination or precipitation. Biological contamination of dispensing equipment, containers, or reagents may lead to inaccurate results.

TEST PROCEDURE


Determining your vaginal pH level is quick, easy, and completely painless. Simply relax and follow the steps below carefully.

1. Wash your hands thoroughly before starting the test.
2. Open the package, take out a test, and remove the foil pouch. Only touch the applicator by its round handle.
3. Hold the applicator by the handle between your thumb and index finger, as shown in the image below.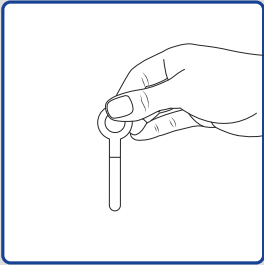


The thumb should always be placed on the side of the vaginal applicator opposite the pH measurement area. Using the applicator in this manner ensures that each test is conducted accurately.

4. Avoid allowing the pH measurement area to come into contact with any other liquids.
5. Sit with your knees apart or place one foot on the toilet seat or another stable surface.
6. Use your free hand to gently spread the vaginal opening, carefully separating the outer labia.
7. Gently insert the vaginal applicator into the vagina, similar to how you would insert a tampon (see image below). Insert it approximately 3 cm into the vagina, but no further than the round handle. The insertion of the vaginal applicator is completely painless.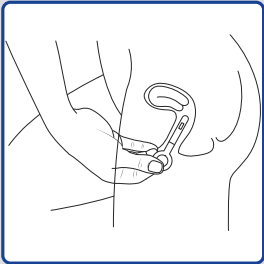


8. Gently press the inserted tube against the back wall of the vagina (toward the rectum) with light pressure for approximately 10 seconds, ensuring that the pH measurement area is well moistened. Adequate moisture is crucial for accurate test results.
9. Carefully remove the vaginal applicator from the vagina.
10. Read the result immediately after removing the vaginal applicator, while the measurement area is still damp. Compare the color in the pH measurement area to the pH color scale and identify the pH-value that best matches your result. Once you have evaluated the test, you can dispose of the used test applicator with the regular household waste.

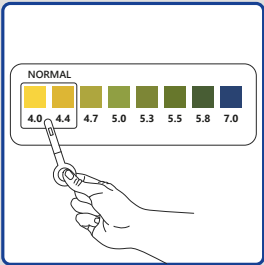
INTERPRETATION OF THE RESULTS

Use the color scale to interpret the test result.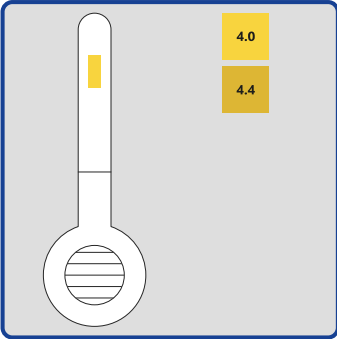

THE PH VALUE OF THE VAGINA IS NORMAL (4.0-4.4)

Vaginal pH values in the range of 4.0 - 4.4 are normal. The measuring area of the applicator is yellowish.

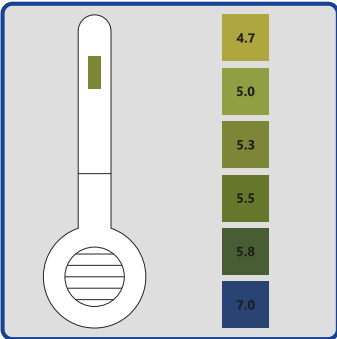THE PH VALUE OF THE VAGINA IS OUTSIDE THE NORMAL RANGE (>4.4)

A vaginal pH value higher than 4.4 is outside the normal pH range. The measuring area of the applicator is darker and turns blue-greenish.

TABLE FOR TEST RESULTS

Test day	1	2	3	4	5
pH level					

IMPORTANT: If the pH measurement does not display a uniform color after testing, it may indicate that the test was not performed correctly. One possible reason is that the pH measurement zone was not adequately moistened with vaginal secretions. In this case, you may repeat the measurement using the same vaginal applicator or opt for a new test.

WHAT TO DO IF THE PH LEVEL IS OUTSIDE THE NORMAL RANGE

If your pH level falls outside the normal range on a single occasion, there is no immediate cause for concern. In such cases, simply repeat the measurement a few hours later or the following day. If the pH level remains outside the normal range, it is advisable to discuss the result with your doctor to determine the cause of the elevated pH level. Keep in mind that various factors can temporarily affect vaginal pH without indicating an underlying medical condition (see also "Precautions").

If you experience any of the following symptoms and your vaginal pH level remains consistently high, you should always consult your doctor:

- Bleeding, including spotting
- Suspected release of amniotic fluid
- Symptoms similar to menstruation
- Fever
- Burning during urination
- Itching or burning in the vagina
- Increased discharge
- Fishy, unpleasant odor

LIMITATIONS OF THE TEST

- This test is intended only for women with regular menstrual cycles (periodic vaginal bleeding).
- If you are pregnant, consult your healthcare provider before using this test, as interpreting results during pregnancy

requires professional expertise.

- Do not perform the test within one day before or five days after your period.
- Do not use the test if you are experiencing menstruation or any form of vaginal bleeding.
- Avoid testing within 48 hours of sexual intercourse or vaginal douching.
- This test is not suitable for individuals who are perimenopausal or menopausal and not undergoing hormonal therapy.
- Several factors can influence vaginal acidity (pH) levels, including blood, semen, menopause, perimenopause (low estrogen), bacterial infections, and Trichomoniasis.
- Do not use the test while sweating (e.g., during or immediately after exercise).
- Wait at least 72 hours after applying vaginal products such as contraceptive creams before using this test. Antifungal treatments do not interfere with the results.

PERFORMANCE CHARACTERISTICS

MEASURING RANGE

The Vaginal pH Test Device has a measuring range of pH 4.0 to 7.0.

TEST ATTRIBUTES: SENSITIVITY AND SPECIFICITY

The Vaginal pH Test Device was evaluated against a standard pH meter. The test demonstrated 100% sensitivity and 100% specificity.

SUITABILITY FOR USE

A lay-user study was conducted to assess the performance of the Vaginal pH Test Device. A total of 100 specimens were tested by 100 non-professional individuals. The study included 43 negative specimens (vaginal pH levels ranging from 4.0 to 4.4) and 57 positive specimens (vaginal pH levels between 4.5 and 7.0). Based on the study results, the Vaginal pH Test Device demonstrated excellent performance with an overall accuracy of 100%.

INTERFERENCE EVALUATION

Various medicinal products were tested as potential interference factors. None of the tested products, at the specified concentrations, interfered with the detection of positive or negative control samples:

Medicinal Product	Concentration (w/v)
K-Y Jelly	20%
Replens	20%
Delfen	20%
Concentrol	20%
Gyne-Lotrim-3	20%
Vagistat-1	1.20%
Massengil	2%
Summer's Eve	20%
Vagisil	20%
Vaginex	20%
Vagisil Powder	4%

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SYMBOLS

 Do not reuse	 In vitro diagnostic medical device
 Temperature limitations	 Consult instructions before use
 Manufacturer	 Authorized Representative in the European Union
 Batch Code	 Use by

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Assure Tech (Hangzhou) Co., Ltd.
Lotus NL B.V.