LuxmedIQ

## VITAMIN-D-SELBSTTEST

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### LITERATURVERZEICHNIS

- Pludowski, P., et al., Guidelines for Preventing and Treating Vitamin D Deficiency: 2023 Update in Poland. *Nutrients*, 2023; 15(3).
- Vitamin D Deficiency – Management Guidelines for Adults and Children (SWL).
- Demay, M.B., et al., Vitamin D for the Prevention of Disease: An Endocrine Society Clinical Practice Guideline. *J Clin Endocrinol Metab*, 2024; 109(8): 1907–1947.

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
**Azubio Diagnostics Co., Ltd.**  
No.219, Guangzhushan Road, Dipu Street,  
Anji County, Huzhou, 313399 Zhejiang, China.  
E-Mail: contact@azubio.com

**CMC Medical Devices & Drugs S.L.**  
C/ Horacio Lengua Nº 18, CP 29006, Málaga, Spain  
E-Mail: info@cmcmedicaldevices.com

**STERILE LANZETTE**

**Sterilance Medical (Suzhou) Inc.**  
No. 168 Putuo Shan Road, New District, 215153 Suzhou, Jiangsu  
People's Republic of China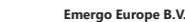

**Emergo Europe B.V.**  
Westervoortsejdijk 60  
6827 AT Arnhem  
The Netherlands

Number: H10110763800  
Version: 1.0  
Effective date: 2026-01-09

LuxmedIQ

CE 0344


# VITAMIN D HOME TEST

For self-testing use

### Please read the instructions carefully before performing the test.

1. Wash your hands with soap and rinse them with clean, warm water.
2. Check the expiration date on the box. Do not use the kit if the packaging is damaged or the kit has expired.
3. Ensure that the kit is at room temperature for at least 30 minutes before use.
4. Check that all components are intact. Remove the blood collector, buffer tube, sterile lancet, and alcohol pad, and place them near the test device.


### CONTENTS

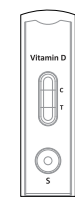Test cassette

Package insert

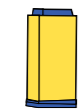
Sterile lancet

Alcohol pad

Buffer tube

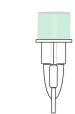Blood collector

Color Card


Materials required but NOT provided: Timer

### TEST PROCEDURE

**Step 1:** Open the pouch and place the test device on a clean, flat surface. For best results, perform the test within one hour of opening the pouch.


**Step 2:** Peel off the aluminum foil seal from the buffer tube.

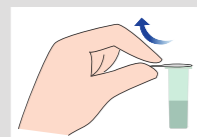
**Step 3:** Clean the puncture site with the alcohol pad provided and allow it to air dry.

**Step 4:** Carefully remove the cap from the sterile lancet. Place the sterile lancet firmly against the puncture site and press until it pricks the finger.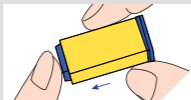

**Step 5:** Keep the finger in an upward position and apply slight pressure to obtain a blood drop.

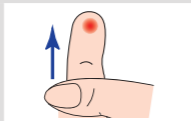
**Note:** Do not over-squeeze the fingers.


**Step 6:** Gently touch the tip of the Blood Collector to the blood droplet. Capillary action will completely fill the Blood Collector with 10 µL of blood and then stop.


**Step 7:** Insert the Blood Collector into the buffer tube and push it firmly to seal.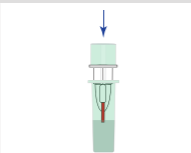


**Step 8:** Gently invert the buffer tube with the Blood Collector 3–4 times to rinse the blood from the collector and mix it thoroughly with the buffer.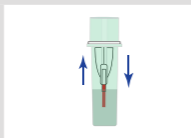

**Step 9:** Remove the cap from the Blood Collector.

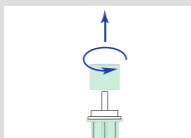
**Step 10:** Invert and gently squeeze 4 drops of solution into the sample well (S), then start the timer.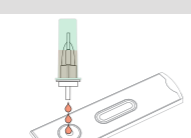

**Step 11:** Wait for the colored line(s) to appear. Read results at 10 minutes. Do not interpret the result after 20 minutes.


### INTERPRETATION OF RESULTS

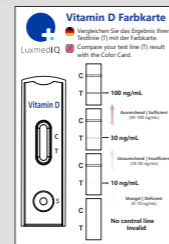Please refer to the provided Color Card to interpret your test results by comparison with the Color Card.

**Sufficient<sup>(1)</sup>** [30–100 ng/mL (75–250 nmol/L)]

Two colored lines appear. The line intensity in the test region (T) is lighter than the 100 ng/mL line and darker than the 30 ng/mL line depicted on the Color Card provided with the kit.

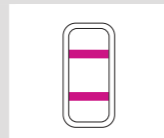
**Insufficient<sup>(1-2)</sup>** [10–30 ng/mL (25–75 nmol/L)]

Two colored lines appear. The line intensity in the test region (T) is lighter than the 30 ng/mL line and darker than the 10 ng/mL line depicted on the Color Card provided with the kit.


**Deficient<sup>(2)</sup>** [0–10 ng/mL (0–25 nmol/L)]

Only one colored line appears in the control region (C). If the result is deficient, it is recommended to consult a physician.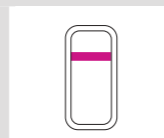

**Invalid**

The control line fails to appear. Insufficient specimen volume or incorrect procedural technique is the most likely cause of control line failure. Please read the instructions carefully and repeat the test. If the result remains invalid, contact your local distributor.

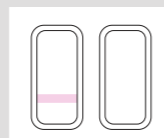### NOTE

1. For vitamin D status categorization, refer to [1] and [2] in the "Literature References" section. Note that other guidelines [3] may define the vitamin D status zone differently.
2. If the line intensity in the test region (T) is darker than the 100 ng/mL line, this indicates that the vitamin D level is greater than 100 ng/mL. It is recommended that vitamin D concentration results be interpreted in accordance with local healthcare practice.

### INTENDED PURPOSE

The Vitamin D Home Test is a rapid chromatographic immunoassay for the semi-quantitative detection of 25-hydroxyvitamin D (25(OH)D) in human fingerstick whole blood. The test is an in vitro diagnostic device intended to be used as a screening aid for vitamin D deficiency. It is intended for self-testing by individuals aged 5 years and over who are suspected of being vitamin D deficient. Children under 18 years of age must be supervised or assisted by an adult when performing the test. The test procedure is not automated. The test does not replace laboratory testing or clinical evaluation for establishing a diagnosis; consultation with a physician for medical assessment is required.

### WARNINGS AND PRECAUTIONS

- For in vitro diagnostic use only.
- For self-testing use only.
- Read all instructions carefully before performing the test. Failure to follow the instructions may result in inaccurate test results.
- The test components are for single-use only. Do not reuse any components, especially the Blood Collector and sterile lancet, to avoid cross-contamination.
- Avoid contact of the buffer with eyes, skin, and mucous membranes. In case of contact, rinse thoroughly with plenty of water.
- Read the results under natural light using the color card.
- Keep out of reach of children. Small test components may pose a choking hazard.
- Use only the supplied test components. Do not replace the buffer with any other liquid. Do not mix or interchange test components from different lots.
- Do not use the extraction buffer if it becomes discolored or cloudy, as this may indicate microbial contamination.
- It is recommended to apply 4 drops of the mixed solution to the sample well; do not use fewer than 3 drops or more than 6 drops.
- Do not touch the membrane of the test device before testing.
- Do not eat, drink, or smoke in the area where specimens or kits are handled.
- Dispose of used kit components in household waste.
- This test kit is intended to be used as a preliminary test only. Repeatedly abnormal results should be discussed with a doctor or other healthcare professional.
- This kit is made from materials that are safe for use at home. The components are not known to cause skin sensitization or allergic reactions.

### STORAGE AND STABILITY

- Store the kit at 2–30°C until the expiry date printed on the sealed pouch.
- Keep the kit away from direct sunlight, moisture, and heat.
- Keep the test in the sealed pouch until use.
- Do not freeze the test kit.
- Open the pouch shortly before the test. Use the test kit within 1 hour of opening the pouch.
- Kit contents are stable until the expiration date marked on their outer packaging.
- Kit buffers are stable until the expiration date marked on the buffer tube.

### INCIDENT REPORT

If any of the following serious incidents occur in relation to the device, the manufacturer and the competent authority of the Member State in which the user and/or patient is established must be informed:

1. The death of a patient, user, or other person;
2. The temporary or permanent serious deterioration in the state of health of a patient, user, or other person;
3. Any serious public health threat.

### LIMITATIONS

1. This self-test only provides an estimated range of vitamin D levels and cannot be used as the sole basis for medical decisions.
2. This test is designed to be used with a fingerstick blood sample. Other body fluids must not be used.
3. The safety and effectiveness of this test have not been established for individuals under 5 years of age or over 94 years of age. Please consult a healthcare professional if you fall outside the intended age range.

### QUESTIONS AND ANSWERS

#### 1. HOW DOES THE VITAMIN D HOME TEST WORK?

The Vitamin D Home Test detects 25-OH VD through visual interpretation of color development on the internal device. During testing, the mixture migrates upward on the membrane chromatographically by capillary action. The membrane is pre-coated with anti-25-OH Vitamin D antibody on the test line region of the strip. As the test sample flows through the membrane within the test device, the colored anti-25-OH Vitamin D antibody-colloidal gold conjugate complexes with 25-OH Vitamin D from the sample. This complex moves further on the membrane by capillary action to the test region (T), where it is immobilized by another anti-25-OH Vitamin D antibody pre-coated on the membrane, leading to the formation of a colored band. The intensity of the colored band in the test line region is 25-OH Vitamin D concentration dependent; the higher the concentration of 25-OH Vitamin D in the tested sample, the stronger the colored band is. A control line present in the test window works as a procedural control. This colored band should always appear in the control line region (C) if the test device is stored in good condition and the test is performed appropriately.

#### 2. WHAT FACTORS ARE RELATED TO THE CONCENTRATION OF VITAMIN D IN THE HUMAN BODY?

- Your vitamin D status and its clinical significance are not static; they can be influenced by several factors, including:
- Age and life stage: Infants, children, adolescents, pregnant or lactating women, and older adults may have different vitamin D requirements and optimal levels.
  - Ethnicity and skin pigmentation: Melanin in the skin reduces the ability to produce vitamin D from sunlight.
  - Geographic location and season: Latitude, time of year, and climate significantly affect sun exposure and thus vitamin D synthesis.
  - Body weight and health status: Conditions such as obesity, malabsorption disorders, liver or renal diseases, and the use of certain medications can influence vitamin D metabolism.

#### 3. WHAT SYMPTOMS CAN OCCUR WHEN THERE IS A DEFICIENCY OR EXCESS OF VITAMIN D?

When vitamin D levels are deficient in the human body, this may lead to lower back pain, loss of height, spinal deformities, increased susceptibility to fractures, and possible changes in mood or sleep patterns. When vitamin D levels are too high (>100 ng/mL), symptoms may include nausea, vomiting, constipation, loss of appetite, excessive thirst, frequent urination, abdominal pain, muscle weakness, and confusion. Consistently high vitamin D concentrations might cause vitamin D toxicity and result in elevated calcium levels, i.e., hypercalcemia, which may lead to serious complications if left untreated.

#### 4. CAN THE RESULT BE INACCURATE?

The results are accurate when the instructions are followed carefully. However, the result may be incorrect if the test device gets wet before the test is performed, if the quantity of blood dispensed into the specimen well is insufficient, or if fewer than 3 or more than 6 drops of solution are added. The blood collector provided in the box helps to ensure the collected blood volume is correct.

#### 5. WHEN SHOULD THE TEST BE USED?

##### 5.1 Purpose of this test

The Vitamin D Home Test is a self-use device intended for individuals who suspect they may be at risk of vitamin D deficiency or who have symptoms associated with vitamin D deficiency.

##### 5.2 Who should consider this test?

You may consider this test if you have one or more of the following risk factors or potential signs and symptoms that could be related to low vitamin D levels:

- Lifestyle factors: limited sun exposure (for example, working indoors, living at higher latitudes, or always using sunscreen).
- Physical factors: having darker skin pigmentation, being overweight, or being an older adult.
- Dietary factors: dietary habits low in vitamin D-rich foods (for example, fatty fish, fortified dairy products, eggs).
- Potential signs and symptoms (please note that these are non-specific and may have other causes): persistent fatigue or tiredness; general muscle weakness or bone discomfort.

**Important Note:** The symptoms above are non-specific and can be caused by many other conditions. If you are experiencing severe or persistent symptoms, you should consult a healthcare professional regardless of your test result.

##### 5.3 Who is this test NOT for?

Current clinical evidence only supports the use of the test in individuals who are suspected of being vitamin D deficient but otherwise healthy. It is not recommended for use without medical supervision in the following special populations: children under 5 years of age, pediatric patients, pregnant or breastfeeding women, and individuals with known liver or kidney disease. This test is a diagnostic test and should not be used to monitor conditions such as rickets, osteomalacia, osteoporosis, or any other diseases. In addition, the clinical utility of vitamin D home testing for conditions such as cardiovascular disease, diabetes, or autoimmune disorders has not been established.

#### 6. HOW TO UNDERSTAND YOUR VITAMIN D LEVEL?

Use the table below for a preliminary assessment of your vitamin D status at the time of testing.

Your Result	Categorization <sup>(1-2)</sup>
≥ 100 ng/mL (250 nmol/L)	Excess
30–<100 ng/mL (75–250 nmol/L)	Sufficient
10–<30 ng/mL (25–75 nmol/L)	Insufficient
<10 ng/mL (25 nmol/L)	Deficient

In addition, the clinical interpretation of what constitutes an optimal vitamin D level is not universal and requires professional assessment. You should discuss your result with a healthcare professional, who can evaluate it in the context of your individual health status, possible risk factors, and symptoms.

#### 7. WILL THE RESULT STILL BE RELIABLE IF I READ IT AFTER 20 MINUTES?

No. The result should be read 10 minutes after adding the solution. Results read after 20 minutes are unreliable.

#### 8. WHAT SHOULD I DO IF THE RESULT IS DEFICIENT OR INSUFFICIENT?

If the result is deficient or insufficient, it means that the vitamin D level in your blood is below 30 ng/mL. You should consult a physician or other healthcare professional and show them your test result. They will decide whether additional tests or further evaluation are needed.

#### 9. WHAT DOES IT MEAN IF THE RESULT IS SUFFICIENT?

If the result is sufficient, it means that the vitamin D level is greater than or equal to 30 ng/mL and is within the normal range.

#### 10. WHAT SHOULD I DO IF THE RESULT SHOWS EXCESS?

If your test result is excess, it means that the vitamin D level in your blood is higher than 100 ng/mL. You should consult a healthcare professional promptly to evaluate your overall health.

#### 11. HOW ACCURATE IS THIS TEST?

This test is highly accurate. In studies comparing it with commonly used laboratory methods, it showed more than 94% agreement in classifying vitamin D levels as deficient, insufficient, or sufficient.

### PERFORMANCE

#### 1. LIMIT OF DETECTION

The limit of detection (LOD) of the Vitamin D Home Test has been verified with total 25-OH VD calibrated by LC-MS/MS. The LOD of the Vitamin D Home Test is 10 ng/mL.

#### 2. CLINICAL PERFORMANCE

The performance of the Vitamin D Home Test has been established in comparison with a chemiluminescence test. A total of 362 samples collected from independent subjects were tested in the study. The accuracy of the test is listed below:

Test results	Accuracy
Deficient	95.40%
Insufficient	94.05%
Sufficient	94.48%

#### 3. ANALYTICAL SPECIFICITY

##### 1. Cross-reacting substances

1,25(OH)<sub>2</sub>D<sub>3</sub> and 1,25(OH)<sub>2</sub>D<sub>2</sub> have been tested using the Vitamin D Home Test, and no cross-reactivity was observed.

##### 2. Interfering substances

The following compounds have been tested using the Vitamin D Home Test, and no interference was observed.

Analytes	Concentration	Analytes	Concentration
Human albumin	52 g/L	Rheumatoid factor	50 U/mL
Bilirubin	257 µmol/L	Paricalcitol	24 ng/mL
Hemoglobin	200 g/L	Vitamin B12	513 pmol/L
Cholesterol	5.7 mmol/L	Vitamin C	342 µmol/L
Triglycerides	2.83 mmol/L	Vitamin D2	1000 ng/mL
HAMA	780.73 IU/mL	Vitamin D3	1000 ng/mL

#### TRACEABILITY AND STANDARD MATERIAL

The performance of this product has undergone rigorous clinical validation. Currently, no international reference materials or standardized measurement protocols exist for result classification (0–10 ng/mL, 10–30 ng/mL, 30–100 ng/mL). Both product performance and production release control materials are traceable to the in-house established 25-hydroxyvitamin D standard, validated by LC-MS/MS (gold standard).

#### LITERATURE REFERENCES

1. Pludowski, P., et al., Guidelines for Preventing and Treating Vitamin D Deficiency: 2023 Update in Poland. *Nutrients*, 2023; 15(3).
2. Vitamin D Deficiency – Management Guidelines for Adults and Children (SWL).
3. Demay, M.B., et al., Vitamin D for the Prevention of Disease: An Endocrine Society Clinical Practice Guideline. *J Clin Endocrinol Metab*, 2024; 109(8): 1907–1947.

#### PRODUCTION INFORMATION

SIZE	1 pc/kit	2 pcs/kit	5 pcs/kit	20 pcs/kit
REF	A70410H11	A70410H12	A70410H15	A70410H20

#### GLOSSARY OF SYMBOLS

	Do not reuse		In vitro diagnostic medical device
	Temperature limitations		Consult instructions before use
	Manufacturer		Authorized Representative in the European Union

	Batch Code		Use by
	CE Mark		Catalog number
	For self-testing		Unique device identifier
	Contains sufficient for <n> tests		Do not use if package is damaged

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**Azubio Diagnostics Co., Ltd.**  
No.219, Guangzhushan Road, Dipu Street,  
Anji County, Huzhou, 313399 Zhejiang, China.  
E-Mail: contact@azubio.com

EC REP

**CMC Medical Devices & Drugs S.L.**  
C/ Horacio Lengo 11º 18, CP 29006, Málaga, Spain  
E-Mail: info@cmcmmedicaldevices.com

STERILE LANCET CE 0123

**SteriLance Medical (Suzhou) Inc.**  
No. 168 Putuo Shan Road, New District, 215153 Suzhou, Jiangsu  
People's Republic of China

**Emergo Europe B.V.**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

Number: H10110763800  
Version: 1.0  
Effective date: 2026-01-09